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This is the future, congratulations with your new Betulacup.

For years we have used sanitary towels, pads, tampons and panty liners. Here you get the healthier, more environmentally conscious, easier and more economical alternative.

Your Betulacup is made of medical silicone approved by the FDA (USA). It is made so that you can easily insert it into the vagina, where it collects menstrual blood. Just as with tampons, you have to get used to using your Betulacup, but it becomes easier after a couple times. Take your time and completely relax the first time you use your Betulacup. Perhaps try and insert it when you are not having your period.

Your Betulacup should be emptied and rinsed every 8-12 hours, however, more often if you find it is necessary.

You can use your Betulacup trouble-free at night, when you play sports or swim (although, empty and rinse your Betulacup after swimming, as a little water can get in the vacuum's holes).

Should you have more questions or need further advice, please feel free to contact us at info@betulacup.com or our website www.betulacup.com

Our contact information

Betula Care

Spangsvej 32, 5210 Odense NV - Denmark

Tlf: +45 24 21 32 34

Email: info@betulacup.com

www.betulacup.com

Betulacup takes no responsibility for the cleaning or protection of cups, nor for problems that occur due to lack of hygiene or incorrect use. The owner has full responsibility for keeping the Betulacup clean.

Before you use your Betulacup

Boil your Betulacup in about 2 L of water for 4-5 minutes, while boiling it can become duller in colour, but this does not affect its effectiveness. Let your Betulacup drain on a piece of kitchen roll. Then it is ready for use.

During your period it is important you rinse it well every time you empty it, and if you are in a public toilet without a sink, wipe your Betulacup with toilet paper and insert it again, but remember to wash it well the next time you empty it.

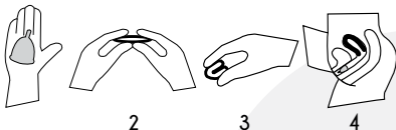
If you feel you need to, you can wash your Betulacup with intimate soap during your period, but remember to thoroughly rinse off any soap residue.

When your period finishes, boil your Betulacup again and store it in the enclosed cotton bag.

Continues on the other side

How to insert your Betulacup

1. Always wash your hands thoroughly with soap and then clean water, both before and after using your Betulacup. If you are not near a sink, use antibacterial gel for your hands.
2. Find a comfortable position, where you feel it is easiest to insert it and completely relax your muscles. Some find it easier to put a foot on the edge of the toilet.
3. Fold your Betulacup twice as shown in the drawing and hold it firmly with 2-3 fingers.
With your free hand, spread your labia, so you can gently insert your Betulacup into the vagina, it unfolds when you release it.
4. If you feel uncomfortable, or the cup is not positioned correctly, start all over again, just like a tampon it can require some practice to insert your Betulacup.

This is how to remove your Betulacup

Empty your Betulacup every 8-12 hours, or more frequently if needed.

1. Wash your hands and thoroughly rinse off any soap. If you are not near a sink, use antibacterial gel for your hands.
2. Find your own comfortable position.
3. Take careful hold of the 'teat' on your Betulacup, relax your muscles and slowly pull the cup out, and then empty it into the toilet.
4. Rinse your Betulacup well in clean water and insert it again if you need to.

It is not risky to use Betulacup when you use it correctly.

Important

Remove your Betulacup before you have sexual intercourse.

Your Betulacup does not protect you against pregnancy.

Betulacup cannot be used as protection against venereal diseases and other sexually transmitted diseases.

Remove your Betulacup if you feel discomfort or pain.

TSS (Toxic Shock Syndrome)

Menstrual cups have never been linked with the disease TSS as tampons have.

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